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•• PRECAUTIONARY MEASURES

- 1. During the work with the material use special glasses, gloves, clothes and mask Goggles are recommended for the patients also.
- 2. Observe the security measures provided during the work with powdery, siliceous substances. Use goggles, a mask and gloves.

•• CONTRAINDICATIONS

Allergic reactions in the anamnesis of the patient especially on methacrylic resins or on any other component of dental materials.

•• COLLATERAL REACTIONS

The product can cause irritation of eyes, skin and mucous membranes. (see. CAUTION section).

•• INTERACTION WITH OTHER DENTAL MATERIALS

Use of eugenol-containing materials in a combination with Jen-Radiance is contraindicated. Eugenol-containing dental materials can have negative impact on ability to polymerization.

•• WARRANTY

The manufacturer warrants the quality of manufactured products. The adverse events inflicted by violation of user manual, storage conditions and other events inflicted by non-stipulated usage of the material are not the subjects of warranty. The customer is responsible for determination of suitability of this product for user's application. Warranty conditions: the product does not comply with requirements declared by manufacturer. In this case the manufacturer replaces the defective material within warranty period.

•• LIMITATION OF LIABILITY

The manufacturer's liability is limited by only cases stipulated by direct legislation of the country.

•• STORAGE

- Inappropriate storage conditions will reduce shelf life and can lead to deterioration of properties of the material. Don't expose this product to direct sunlight. Store a material in a dry place.
- Store this product at the temperature of 4 - 25 °C [39.2 - 77 °F].
- DO NOT FREEZE!
- Shelf life of Jen-Radiance is 3 years.

•• RECYCLING

Dispose of the medical device in accordance with local / regional / national / international legal requirements.

•• PACKING

Jen-Radiance is packed in syringes on 4 g.

- Translucent (incisal) shades: I (Incisal - Enamel Normal), IOP (Incisal Opaline - Enamel White Opalescent);
- Universal dentin shades: A1, A2, A3, A3.5, B1, B2, B3, C2, D2, BW (Bleach White), XBW (Extra Bleach White);
- Opaque shades: A1-O, A2-O, A3-O, A3.5-O, UO (Universal Opaquer), SWOP (Super White Opaquer).

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